4. Apply **BIOPHAT** straight to the places indicated using a brush;
5. Instruct the patient not to eat solid food during the next 4 hours after application;
6. In the same day after applying the product it is recommended not to brush the teeth in order to avoid varnish film elimination.

NOTE: The product is highly adhesive, therefore no etching is required.

Recommended period of permanence in the mouth: it is until 12 hours.

It is recommended to do 2 or 3 applications at every 3 or 4 days. After applying, do not brush the teeth, allowing the product acting as longer as possible.

#### **PRECAUTIONS FOR A SAFE USE:**

- This product is topic - it does not have to be ingested.
- On the BIOPHAT application day, do not do any other form of preventive treatment using fluorides (a.e. topical application of fluoride gel or fluoride solution).

#### **SPECIAL CARES:**

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

#### **CONTRAINDICATION:**

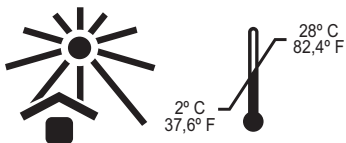
**BIOPHAT** should not be used in patients is known to have sensibility to any of the ingredients listed.

#### **WARNINGS:**

Do not use material after expiry date.

The manufacture is not responsible for any damage resulting from a misuse or not provided for the instructions of use. "KEEP MATERIAL OUT OF THE REACH OF CHILDREN".

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Reg. ANVISA: 10298550054.

## **COMPOSITION:**

Sodium Fluoride, Calcium Fluoride, Glycerin, Vehicle, Pigment and Flavor.

## **INDICATION:**

For prevention and prophylaxis of decay where there are demineralization areas of the teeth.

On the treatment of hypersensitive dentinary colon, mainly after scaling procedures.

To avoid secondary marginal decay (apply product in the edges of the prepare after restorations, incrustations, crowns and bridges). To prevent decays in patients wearing orthodontics appliance, applying product in the edges of the brackets. After removing brackets, apply product in demineralized areas. After bleaching apply the product to remineralize the teeth.

In special patients who has no conditions to do a proper cleansing.

NOTE: It does not interfere in bleaching final result.

## **TECHNICAL INFORMATION:**

**BIOPHAT** contains Sodium Fluoride and Calcium Fluoride and it is indicated to be applied in demineralized areas of the teeth. The vehicle used is based on natural resins which guarantees easy manipulation and the characteristic to be highly adhesive to enamel surface, granting higher period of time in the mouth. Presence of two Fluoride sources guarantees a quicker initial action (sodium fluoride) and a longer but deeper action (calcium fluoride).

In hyperesthetic dentin areas **BIOPHAT** will act as filling the dentinal canals not allowing external stimulation (which provoke pain), to get through.

Fluoride concentration present in the product:

6% Sodium fluoride = 2,71% fluoride ions

6% Calcium fluoride = 2,92% fluoride ions

## **USE INSTRUCTIONS:**

1. Clean tooth surface where there is demineralization or areas where prevention is desirable;
2. Remove excess of teeth humidity before application;
3. It is recommended to begin application by inferior arch to avoid that saliva gets the teeth humid;