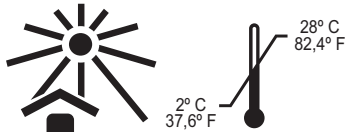
EVIPLAC

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EVIPLAC

Disclosing Solution of Bacterial Plaque

biodinámica

CONTENTS:

1x10mL flask **EVIPLAC**
Reg. ANVISA: 10298550044.

COMPOSITION:

Basic Fucsin 0,7%; Alcohol 96º and Deionized Water.

INDICATION:

EVIPLAC is a solution indicated for disclosing the bacterial plaque.

TECHNICAL INFORMATION:

EVIPLAC based on Basic Fucsin guarantees to the product a higher performance for disclosing the bacterial plaque, on apparently clean teeth, staining them with a color that contrast to the teeth. Such device not only disclose the plaque but helps in the cleansing instruction as well.

USE INSTRUCTIONS:

I - Topic Application:

1. Soak a cotton tail or cotton ball with the product and apply on all dental surfaces;
2. After, rinse with water to remove excess.

II - Mouth rinse:

1. Dispense in a glass approximately 6ml of water, add 1ml (approximately 16 drops) of **EVIPLAC**, rinse for one minute and spit; (Do not forget to clean the basin well with running water to avoid staining).
2. After, rinse with water to remove excess.

NOTE:

All the areas stained indicate bacterial plaque presence that must be completely removed by toothbrush and dental floss.

Can be daily used and preferably before the last tooth brushing.

SPECIAL CARES:

The main use of the product is to color the bacterial plaque of a contrasting color with teeth and tissue.

It must be used only as an assistant in the disclosing of the bacterial plaque.

Disclosed plaque must be removed when brushing it.

Protect clothes away from the contact with the product once the product's stains are difficult to be removed.

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINDICATION:

EVIPLAC should not be used in patients is known to have sensibility to any of the ingredients listed.

WARNINGS:

Do not use material after expiry date.

The manufacture is not responsible for any damage resulting from a misuse or not provided for the instructions of use.

“KEEP MATERIAL OUT OF THE REACH OF CHILDREN”.

FOR USE IN DENTISTRY ONLY