proceeds the complete return from the tooth in the clockwise one).

3. If necessary, to facilitate the removal, slightly humidify the retraction cord with water.

Note: Do not remove the cord until immediately before impression taking.

IDEAL TIME OF PERMANENCE OF THE PRODUCT:

When do not using hemostatic solution, the ideal time of permanence of the product for the attainment of the retraction is at least 5 minutes.

When using hemostatic solution, follow the time indicated in the manufacture's instructions.

When moldings, leave the cord located until immediately before impression taking.

PRECAUTIONS FOR A SECURE USE:

This product only should be used on a healthful tissue, free from wounds, bleedings or infections.

If the gingival retraction is necessary for several teeth, do the gingival retraction procedure on each tooth.

When using a hemostatic/retractive solution, follow the solution's manufacturer instructions for the impregnation.

In case of cervical lesion, the cord should be inserted before the cavity preparation.

When the preparation involves the proximal areas, maybe necessary to use two retraction cords, in order to get a better reproduction of the gingival margin, once is more difficult to retract the interdental gingiva.

After a procedure using the retraction cord, inspect the gingival sulcus to verify if had happened any lesion and to certify that there are no retraction cords fragments on the gingival cervix, avoiding that a strange body causes a local inflammation.

SPECIAL CARES:

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINICATION:

RETRAFLEX should not be used in patients is known to have sensibility to the product.

WARNINGS:

Do not use material after expiry date.

The manufacture is not responsible for any damage resulting from a misuse or not provided for the instructions of use.

"KEEP MATERIAL OUT OF THE REACH OF CHILDREN".

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100% cotton cord.

INDICATION:

RETRAFLEX is indicated to retract the gum in some dental procedures, optimizing the results when effecting dental impressions, facilitating the access and the visibility of all the tissue where is operating, besides help to prevent damages in the tissue gum during the confection of the cavitaries procedures.

TECHNICAL INFORMATION:

RETRAFLEX is a cord made of cotton for gum retraction in some dental procedures, more common for moldings, and in the necessity to move away the gum tissue for restoration confections.

The product action is mechanics, through its interposition between the free gum edge and the dental surface. The humidity of the normal tissue, water or solutions for gum retraction makes the twisted cord promote the retraction of the tissue gum.

USE INSTRUCTIONS:

1. Cut a piece of **RETRAFLEX** in the desired length or use the individual packing (Uni Dose).

Note: The cord could be absorbed in hemostatic solution to get a bigger retraction.

2. Place the cord around the neck of the tooth, and introduce in the ridge compressing carefully using a spatula without the cutting extremity. (Initiate for the interproximal space where the ridge generally is deeper and then